# DSF 130

**Duschstuhl, faltbar**  
**Shower Chair, foldable**

ART. NR. / REF: 550 200 100

CE

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Dear users,

we congratulate on the buying of a technically matured and the best available technology product.

This instructions helps you and the careers as a basis for sure, safe use of the Shower chair and a sensible use of the product possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

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Standing insecurity / standing inability with use decrease / of missing functional ability of the lower extremities or other illnesses who are affecting the stability (e.g., strength loss with evil illnesses, chronic, advanced heart circulatory illnesses and lung illnesses) infers with a body position weakness, so that an independent use of the shower is still allowed.

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This product is suitable for reapplication. With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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
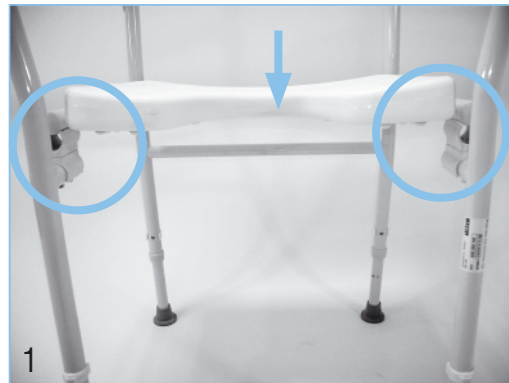
The shower chair is preassembled, the backrest must be only be latched in and the 4 feet must be assembled (see next chapter).

Unfold him as illustrated and put the seat upwards.

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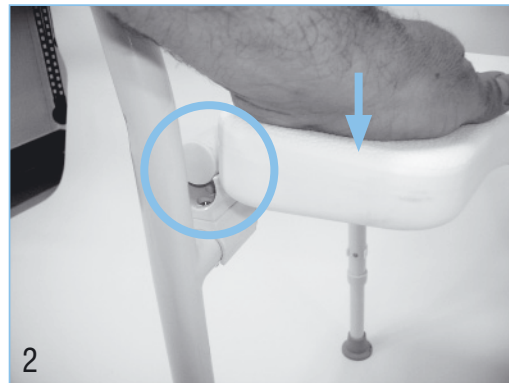Swing both side parts forwards and then the seat record down them on the hold clips lies (picture 1).

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Press now the seat plate hard down until it engages audible in the hold clips on the left and on the right side (picture 2).

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
Check the correct position of the seat plate in the hold clips. (Picture 3).

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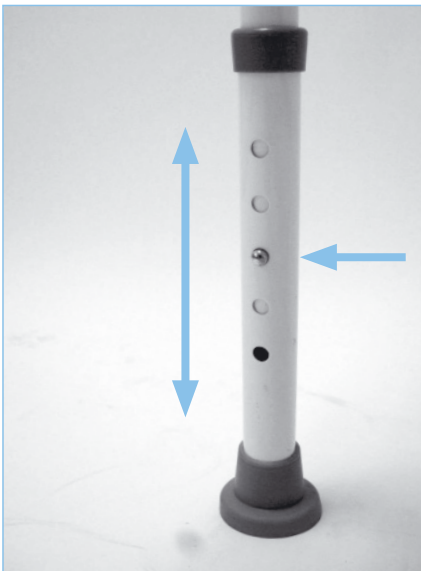To adjust the footrests in the height press the brass pin. Then the footrests can be shifted in 5 steps in the height.

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- 1 Product name
- 2 Article number
- 3 Serial number
- 4 Date of manufacturing year/month
- 5 Attention: See operating instructions

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